

LATERA absorbable nasal implant

POST-PROCEDURE REMINDER



Don J Beasley, MD
Camille Buchmiller, PA-C
208-229-2368

Patient Name: _____ Implantation date: _____

You have received the LATERA absorbable nasal implant. The implant supports the upper and lower lateral nasal cartilage and absorbs over a period of approximately 18 months.

Your doctor spoke with you about the importance of not manipulating your nose during the healing period. A reminder of that discussion is below.

Recovery period will vary based on the individual procedure and the patient. Many people do not have much discomfort but every patient is different. As with any nasal procedure, you may experience some temporary swelling, which could have a positive or negative impact on your breathing.

Contact your doctor if you have any questions, concerns or comments about your implant.

Week 1	Week 2	Weeks 3 and 4
Do not pinch or blow your nose		
Avoid moving your nose side to side		
Avoid aggressive or unnecessary manipulation of your nose		
Avoid strenuous activity		
Sneeze with your mouth open		
Do not place a tissue or other object inside your nose		

Notes:

ENT

Instructions for Use: The LATERA absorbable nasal implant is indicated for supporting upper and lower lateral nasal cartilage.

Please see Instructions for Use (IFU) for a complete listing of warnings, precautions, and adverse events.

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

This document is intended solely for the use of healthcare professionals. A surgeon must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. We do not dispense medical advice and recommend that surgeons be trained in the use of any particular product before using it in surgery.

The information presented is intended to demonstrate Stryker's products. A surgeon must always refer to the package insert, product label and/or instructions for use, including the instructions for cleaning and sterilization (if applicable), before using any of Stryker's products. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your representative if you have questions about the availability of Stryker's products in your area.

Stryker or its affiliated entities own, use, or have applied for the following trademarks or service marks: LATERA and Stryker. All other trademarks are trademarks of their respective owners or holders.

The absence of a product, feature, or service name, or logo from this list does not constitute a waiver of Stryker's trademark or other intellectual property rights concerning that name or logo.

MKT20756 rD 10/2018

Copyright © 2018 Stryker